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## Medical device (CE)

Please read information and directions for use carefully!

### Effects:

- fast help with erectile dysfunction
- maintain longer lasting erections
- intensifies orgasm
- medical device

### Areas of application:

Constriction rings are designed to prevent premature draining of blood from the penis (so-called venous leak or veno-occlusive dysfunction) in order to maintain, or encourage a stronger erection. The purpose of a constriction ring is to obstruct the outflow of blood inside the penis by evenly spread compression in order to maintain, significantly strengthen, and/or stretch the erection. In order to do this it must be placed at the base of the penis.

### Application:

The medical grade silicone ring is simply stretched at the beginning around the penis, at the base (depending on the ring the size can be adjusted according to personal needs). Simple but very effective, the ring helps to form a stronger and longer erection. Start with a more loose fit or a bigger size and gradually increase the tightness until the effect is noticeable while not feeling uncomfortable.

### Attention!

Please note that the ring should not be worn for any longer than 30 minutes at a time, otherwise it can lead to circulatory problems or loss of rigidity.

### Side effects:

Depending on the tightness, the ring can lead to numbness, coldness, or slight bruising in the genitalia. If you experience any of these symptoms or any pain or discomfort while using the ring, you should remove the device right away.

### Hygiene:

The constriction ring should be thoroughly washed after each use with soap and hot water. For best results, we recommend JOYDIVISION „clean'n'safe“ for care and disinfection.

### Note:

For prolonged erection disorder you should consult a physician.

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